Duplica^{RealTime}Factor II G20210A Genotyping Kit

REV: EER001032_IFU_REV.00D_ENITA

REF: EER001032 -32 tests

Instructions For Use

INTENDED USE

Duplica^{RealTime}Factor II G20210A Genotyping Kit is an *in vitro* nucleic acid amplification test for the detection of the nucleotide substitution G20210A (allelic variant g.46739505G>A) of the blood coagulation Factor II gene, in human genomic DNA, extracted from peripheral whole blood samples, collected in EDTA.

INTRODUCTION

Prothrombin G20210A (also the Prothrombin 20210 mutation, the Factor II mutation, or the Prothrombin mutation) is a genetic variant that approximately doubles or triples the risk of forming blood clots in the veins. The variant is commonly associated with the Venous ThromboEmbolism (VTE) disease, which includes both deep vein thrombosis and pulmonary embolism. Most carriers, though, never develop VTE in their lifetime. About 2 to 3% of Caucasians carry the Prothrombin G20210A variant, and it confers a 2 to 3 fold higher risk of VTE. Deficiencies in the anticoagulants Protein C and Protein S give a higher risk (5 to 10 fold). Behind non-O blood type and Factor V Leiden, Prothrombin G20210A is one of the most common genetic risk factors for VTE. The variant causes elevated plasma Prothrombin levels (hyperprothrombinemia), possibly due to increased pre-mRNA stability. Prothrombin is the precursor of Thrombin, which plays a key role in causing blood coagulation. G20210A can thus contribute to a state of hypercoagulability, but not particularly with arterial thrombosis. Recently it has been showed a 1.3 fold increased risk only for coronary disease. Heterozygous carriers who take oral contraceptives are at a 15 fold increased risk of VTE, while carriers also heterozygous for Factor V Leiden have an approximate 20 fold higher risk and it seems to be increased when FII G20210A transition is associated with the Factor V Leiden mutation. In a recommendation statement on VTE, genetic testing for G20210A in adults that developed unprovoked VTE was disadvised, as was testing in asymptomatic family members related to G20210A carriers who developed VTE. In those who develop VTE, the results of thrombophilia tests (wherein the variant can be detected) rarely play a role in the length of treatment.

PRINCIPLE OF THE TEST

Duplica^{RealTime}Factor II G20210A Genotyping Kit is designed to identify the G20210A nucleotide substitution of the blood coagulation Factors II gene. Two reaction mixes are provided for the amplification:

- **AMPLIFICATION MIX:** with Hot Start Taq DNA polymerase, nucleotides, MgCl₂ and buffer.
- **OLIGO MIX:** with primers and fluorogenic probes.

Duplica^{RealTime}Factor II G20210A Genotyping Kit is based on specific recognition and amplification of target sequences by PCR, and the simultaneous detection of the accumulation of PCR amplification products by fluorescent DNA probes. In particular, the probe designed to detect the Wild Type allele carries the fluorophore FAM (6-carboxy-fluorescein) at the 5' end, while the probe detecting the Mutated allele, is labelled with the fluorophore HEX (hexachloro-fluorescein). Both the probes have a non fluorescent black quencher at the 3' end. If excited, the whole probe doesn't emit fluorescence, since the proximity of the quencher to the reporter prevents the emission of the fluorescence from the reporter (quenching effect).

REAGENTS PROVIDED

Each kit contains enough reagents to perform 32 tests when used in 4 analytical sessions with 5 **samples**, 1 **Wild Type Allele Control (Control 1, C1)**, 1 **Mutated Allele Control (Control 2, C2)** and 1 **Reaction Blank (BM)** each.

Kit Components

Reagent	Color Code	Storage (range, °C)	Volume (µl)	Quantity (tubes)
Oligo Mix (OM)*	Green Cap	-22÷-18	400	1
Amplification Mix (AM)	Blue Cap	-22÷-18	400	1
Control 1 (C1, Wild Type)	Red Cap	-22÷-18	>50	1
Control 2 (C2, Mutated)	Yellow Cap	-22÷-18	>50	1
Reaction Blank (BM)	White Cap	-22÷-18	>50	1

*protect the tube from direct light

STORAGE AND HANDLING

All reagents must be stored at **-22÷-18°C** and can be used until the expiry date printed on the labels. Do not freeze and thaw the products more than six times.

MATERIAL REQUIRED BUT NOT PROVIDED

- Extraction kit for DNA purification (refer to the specific handbook's section)
- Optical tubes (Smart Tubes) for SmartCycler® Real Time PCR platform
- Disposable powder-free gloves and laboratory coat
- Variable volumes pipettes (5-20 µl, 20-200 µl, 100-1000 µl)
- Disposable tips with DNase/RNase-free aerosol barriers
- Tube racks
- Desktop centrifuge
- Mini Centrifuge with Smart tubes holder
- PCR box
- Refrigerator
- Deep-freezer
- Thermocycler for Real Time PCR

The kit has been developed to be used on Real Time PCR SmartCycler® (Cepheid) platform. Other makes and models should be fully tested and evaluated for optimal performance by the user before reporting results.

The equipments should be regularly maintained, in accordance with the manufacturer's instructions, and calibrated to ensure accurate PCR cycling and optimal performance.

PRECAUTIONS AND WARNINGS

- In compliance with Good Laboratory Practice, define three separate laboratory's areas for: DNA extraction, PCR reaction mix preparation; manipulation of controls provided with the kit. Each area must have dedicated pipettes and laminar flow hood.
- If required, Euroclone Diagnostica offers the necessary technical support for the correct use of the kit.
- Carefully read this Instruction for Use before using the kit
- Do not use the reagents after the expiry date
- Thaw and carefully mix the reagents of the kit before use
- Do not mix the reagents from different lots of the product
- Use calibrated and regularly checked pipettes and instrumentation only
- Use dedicated laboratory equipments. Change gloves frequently
- Periodically wipe the working area with 0,5% hypochlorite
- Use powder-free gloves. Do not leave fingerprints on optical part of Smart Tubes.
- Materials containing or potentially-containing infectious agents must always be manipulated in a separated microbiological safety room under a Biohazard biological hood.
- In case of damaged package, contact the technical support before using the kit
- Do not use the product when stored at temperatures other than those indicated on the labels or described in this Instructions For Use.
- In case of spillage of the kit contents, please refer to the specific Material Safety Data Sheet (MSDS, available on request).
- The kit reagents, individual protective equipments, used materials, biological samples and test residuals must be disposed in accordance with local regulations.
- Patient Drug treatment may interfere with the final result of the molecular biology analysis.

OPERATING PROCEDURE

a) DNA purification

For genomic DNA purification EuroClone Diagnostica recommends to use:

-Duplicα Blood DNA kit (ref. EDI002250) and Duplicα NA Body Fluid kit (ref. EDI004200) with Duplicα® PREP Automatic Extractor (ref. EDI001) for automated purification.

For manual extraction/purification:

-Fasst DNA Releaser (ref. EMR057050) for **fresh** peripheral blood (i.e. stored for up to 24 hours at 2÷8°C).

-Spin DNA Purification Kit (ref. EMR061050) for **frozen** blood or stored at 2÷8°C for more than 24 hours.

Other extraction reagents and methods should be fully tested and evaluated for optimal performance by the user before reporting results.

Attention! Use blood samples in EDTA anti-coagulant solution.

b) Thermalcycler Setup

Important points before starting: Refer to SmartCycler® System User's Manual for detailed instructions on how to operate the SmartCycler® System and software version. We recommend to switch on the instrument, and to set the thermal profile before preparing the reaction mix.

N.B.: before starting the run is recommended to save the file as "FACTOR II G20210A"; this way it is possible to save the Thermal Profile and settings and recall them in subsequent runs.

- Open the SmartCycler® software and click "Define Protocols".
- Click "New Protocol" (bottom left).
- Enter Name "FACTOR II G20210A".
- Enter the parameters for the as FACTOR II G20210A outlined in Table below.
- When ready click "Save Protocol".

TIME	TEMPERATURE	CYCLES
10 min	95°C	1
20 sec	95°C	50
30 sec	60°C	Fluorescence Acquisition

c) PCR mix preparation

The total reaction volume is **25 µl**.

For each experiment prepare a PCR mix for the 2 controls (**C1** and **C2**), 1 **Reaction Blank (BM)** and **n+1** samples. The reagents of the PCR mix have to be mixed as indicated in the table below:

REAGENT	VOLUME (µl)
Amplification mix	10
Oligo mix	10
Extracted DNA	5

The PCR mix has to be freshly prepared every time

After its preparation, aliquot **20 µl of Master Mix** in the tubes for PCR then add in each tube **5 µl** (100-250 ng/reaction) from the **extracted DNA** or **control DNA**.

d) Run Assay Setup

- Centrifuge Smart tubes for 10 seconds in the appropriate centrifuge.
- Insert the Smart-Tubes into the required PCR "Sites" (reaction chambers) of the SmartCycler®. Press the tubes firmly and close each PCR Site.
- Click "Create Run" and specify "Run Name". If needed add remarks in "Notes" and Lot Number in "Lot".
- Select the Dye Set: "FCTC25".
- Click the "Add/Remove Sites" button. A new window "Select Protocols and Sites" will be displayed in which you need to specify the "Protocol" and the number of PCR "Sites" for the test procedure. Select "FACTOR II G20210A" as the "Protocol". Highlight the "Sites" in use for the test procedure and press the right arrow. When the Protocol and Sites have been selected click "OK".
- Enter the sample I.D.'s for every site in the Sample ID column.
- Click the "Start Run" button.
- When the run is completed, discard the Smart Tubes according to local regulations.

e) ANALYSIS and INTERPRETATION of RESULTS

Important points before starting: For a detailed description on how to analyze data, refer to SmartCycler® *System User's manual*. **Always visually inspect the amplification plot for each sample tested versus C_T values obtained with the software.**

Results interpretation

The fluorescence in each channel indicates the hybridisation of the allelic specific probes:

- **Channel 1** for **FAM = Wild Type allele probe** and **Channel 2** for **HEX= Mutated allele probe**. If a sample shows a fluorescence in **FAM**, the sample has **Wild Type allele**. If a sample shows a fluorescence in **HEX**, the sample has **Mutated allele**.
- Therefore, if only a **FAM signal** is detected the sample is **Homozygous Wild Type**, whereas if only a **HEX signal** is detected the sample is **Homozygous Mutated**. Finally, if both **FAM** and **HEX** are detected the sample is **Heterozygous**.
- Condition in which no signal is detected indicates PCR inhibition. In this case, refer to troubleshooting.
- Control 1 (Wild Type), Control 2 (Mutant) and reaction blank (B) are provided in order to properly set the threshold line before samples analysis. After the run, the threshold line has to be set so that: **Control 1 is detected in FAM and not detected in HEX, whereas Control 2 is detected in HEX and not detected in FAM. The Reaction Blank must not be detected in any channel.**

If these three conditions have been met, the run is valid and it's possible to analyse the data; otherwise the run is not valid. It's responsibility of the user to validate the run.

Results Interpretation Table

FAM	HEX	Results
Detected	Not detected	Wild Type
Not detected	Detected	Mutated
Detected	Detected	Heterozygous
Not detected	Not detected	Inhibition

TROUBLESHOOTING

Problem 1: Weak or no signal of unknown samples.

1. The PCR was inhibited:
 - Make sure to use a recommended DNA purification method and carefully follow the manufacturer's instructions.
2. Pipetting error due to omitted reagents or samples:
 - Repeat the analysis starting from the PCR.
3. Deterioration of dyes and/or primers. The reagents storage conditions didn't comply with the instructions:
 - Check storage conditions.
4. Very low starting amount and/or low purity of genomic DNA. Improper DNA extraction:
 - Repeat the DNA purification.
5. Wrong channel/filter was chosen. The PCR conditions didn't comply with the instructions:
 - Check the PCR conditions and select the fluorescence channels reported in the protocol for the Unknown Sample detection.

Problem 2: Weak or no signal of the Wild Type Control C1 and/or Mutated Control C2.

1. The PCR conditions didn't comply with the instructions:
 - Check the amplification protocol and select the fluorescence channel reported in the manual.
2. Deterioration of dyes and/or primers. The reagents storage conditions didn't comply with the instructions:
 - Check storage conditions.

Problem 3: Any signal with Reaction Blank BM.

1. Contamination during PCR preparation procedure. All samples results are INVALID:
 - Decontaminate the working area and all instruments.
 - Pipette the controls C1 and C2 at last.
 - Repeat the PCR preparation using a new set of reagents.

Problem 4: HEX signal in Wild Type Control C1 and/or FAM signal in Mutated Control C2.

1. Contamination during PCR preparation procedure. All samples results are INVALID:
 - Decontaminate the working area and all instruments.
 - Use tips with aerosol barriers only. Change tips between tubes/wells.
 - Repeat the PCR preparation using a new set of reagents.

Problem 5: Wide Fluctuations in fluorescence values.

1. The PCR Master Mix is not well prepared:
 - Carefully repeat the PCR preparation procedure.
2. Air bubbles trapped in the PCR tubes:
 - Check the presence of air bubbles before starting a new run.

Problem 6: Absence of any fluorescent signal.

1. Verify the performance of the thermalcycler:
 - Calibrate the equipment.
2. Deterioration of dyes and/or primers. The storage conditions didn't comply with the instructions:
 - Check storage conditions.
 - Check the expiry date of the kit.

Problem 7: The thermalcycler gives an error message.

1. Refer to the Real Time PCR instrument user manual or contact the local technical support of the Real Time PCR instrument company.

Problem 8: The kit reagents left out of the storage range temperature.

1. These reagents must be stored **as indicated**, for a proper execution of the test. The performance of the product is not guaranteed if the reagents have not been properly stored.

"THE PURCHASE OF THIS PRODUCT GRANTS THE PURCHASER RIGHTS UNDER CERTAIN ROCHE PATENTS TO USE IT SOLELY FOR PROVIDING HUMAN IN VITRO DIAGNOSTIC SERVICES. NO GENERAL PATENT OR OTHER LICENSE OF ANY KIND OTHER THAN THIS SPECIFIC RIGHT OF USE FROM PURCHASE IS GRANTED HEREBY"

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EuroClone Diagnostica Srl