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Introduction

External controls **Serazym**[®] Stool diagnostics are used for external quality control of the product lines **Serazym**[®] (Seramun) and ProSpecT[®] or IDEIA[™] (Oxoid). Additionally the controls for the detection of *C. difficile* GDH and toxins can be used for Premier[®] *C. difficile* GDH or Toxins A&B ELISA (Meridian). The controls for *H. pylori* can also be used for Premier[®] Platinum HpSA[®] PLUS ELISA (Meridian). Using the controls, qualitative results are generated (positive or negative).

Packaging Unit

Two External controls **Serazym**[®] Stool diagnostics are offered for every test parameter as a set with 400 µl stabilized stool suspension each. The sets are shipped twice per year on predefined dates. The dates are announced by Virotech Diagnostics GmbH.

Preparation

Allow External controls **Serazym**[®] Stool diagnostics to reach room temperature prior to use and mix thoroughly.

External controls **Serazym**[®] Stool diagnostics are treated like patient samples and are diluted with the respective sample diluent according to the manufacturer's specifications.

Note: The **Serazym**[®] Verotoxin 1+2 external controls do not contain viable bacteria. Therefore, they should not be pre-incubated in enrichment bouillon (potential false negative results by high dilution). The controls should be used like stool samples without enrichment according to manufacturer's specifications.

Intended Use

External controls *Serazym*[®] Stool diagnostics can only be used in the following ELISA kits. There are no reference values for the use in ELISA kits of other manufactures.

Control-Set	Order-No.	ELISA kit	ELISA kit
Rotavirus-1 Rotavirus-2	EK-020	<i>Serazym</i> [®] Rotavirus	ProSpecT [®] Rotavirus
Adenovirus-1 Adenovirus-2	EK-017	<i>Serazym</i> [®] Adenovirus	ProSpecT [®] Adenovirus
Astrovirus-1 Astrovirus-2	EK-045	<i>Serazym</i> [®] Astrovirus	ProSpecT [®] Astrovirus
Norovirus-1 Norovirus-2	EK-061	<i>Serazym</i> [®] Norovirus	IDEIA [™] Norovirus
Campylobacter-1 Campylobacter-2	EK-093	<i>Serazym</i> [®] Campylobacter	ProSpecT [®] Campylobacter
Clostridium difficile GDH-1 Clostridium difficile GDH-2	EK-107	<i>Serazym</i> [®] Clostridium difficile GDH	Premier [®] C. difficile GDH
Clostridium difficile Toxin A+B-1 Clostridium difficile Toxin A+B-2	EK-040	<i>Serazym</i> [®] Clostridium difficile Toxin A+B	ProSpecT [®] C. difficile Toxin A/B Premier [®] Toxins A&B
Verotoxin 1+2-1 Verotoxin 1+2-2	EK-030	<i>Serazym</i> [®] Verotoxin 1+2	ProSpecT [®] Shiga Toxin E. coli
Giardia-1 Giardia-2	EK-106	<i>Serazym</i> [®] Giardia	ProSpecT [®] Giardia
Cryptosporidium parvum-1 Cryptosporidium parvum-2	EK-039	<i>Serazym</i> [®] Cryptosporidium parvum	ProSpecT [®] Cryptosporidium
Entamoeba histolytica-1 Entamoeba histolytica-2	EK-018	<i>Serazym</i> [®] Entamoeba histolytica	ProSpecT [®] E. histolytica
H. pylori 2nd Gen.-1 H. pylori 2nd Gen.-2	EK-114	<i>Serazym</i> [®] H. pylori 2nd Gen.	Premier [®] Platinum HpSA [®] PLUS

Evaluation Of Results

The evaluation of the External controls *Serazym*[®] Stool diagnostics is carried out according to the manufacturer's specifications of the respective test.

Storage And Stability

External controls *Serazym*[®] Stool diagnostics may only be used within the expiry date reported on its respective label (stored at 2...8°C). External controls *Serazym*[®] Stool diagnostics are stable for 4 weeks from time of manufacture.

After expiration of Stability, the qualitative results (positive or negative) are published on the homepage of Virotech Diagnostics GmbH.

For this please enter the internet address <http://www.virotechdiagnostics.com/service.html> or visit www.virotechdiagnostics.com and choose under section „service“ External controls *Serazym*[®] Stool diagnostics. There you will find the certificate of analysis with target values which can be printed out.

Manufacturer

Catalogue-No.

Lot-No.

Number of determinations

Biohazard

Notice advices

Use by

Storage temperature

Consult instructions for use

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Seramun Diagnostica GmbH · Spreenhagener Straße 1 · 15754 Heidesee · Germany · www.seramun.com
Virotech Diagnostics GmbH · Löwenplatz 5 · 65428 Rüsselsheim · Germany · www.virotechdiagnostics.com