## Duplica<sup>RealTime</sup> *Neisseria gonorrhoeae* 2<sup>nd</sup> Generation Detection Kit

REV: EBR018032\_IFU\_REV.00D\_ENITA

REF: EBR018032 -32 tests

### Instructions For Use

#### INTENDED USE

**Duplica<sup>RealTime</sup> *Neisseria gonorrhoeae* 2<sup>nd</sup> Generation Detection Kit** is an *in vitro* nucleic acid amplification test for the detection of *Neisseria gonorrhoeae* in human genomic DNA extracted from clinical specimens or from BD ProbeTec ET™ lisedated samples.

#### INTRODUCTION

*Neisseria gonorrhoeae* is responsible for the Sexually Transmitted Disease (STD) Gonorrhoea. Gonorrhoea is a serious public health problem and is the most commonly reported infectious disease. *Neisseria gonorrhoeae* is a species of Gram-negative kidney bacteria most frequently spread during sexual contact. However, it can also be transmitted from the mother's genital tract to the newborn during birth. In women, the cervix is the most common site of Gonorrhoea, resulting in endocervicitis and urethritis, which can be complicated by pelvic inflammatory disease (PID). In men, Gonorrhoea causes anterior urethritis.

#### PRINCIPLE OF THE TEST

**Duplica<sup>RealTime</sup> *Neisseria gonorrhoeae* 2<sup>nd</sup> Generation Detection Kit** is an assay for the DNA-based detection of *Neisseria gonorrhoeae*. The kit was designed to identify the sequences of **Cryptic plasmid pJD1** and **16S rRNA**. The reagents for the amplification are ready-to-use and provided in 3 separate tubes:

- **AMPLIFICATION MIX:** with Hot Start Taq DNA polymerase, nucleotides, MgCl<sub>2</sub> and buffer.
- **OLIGO MIX:** with primers and fluorogenic probes.
- **INTERNAL CONTROL OF AMPLIFICATION:** to check the successful amplification reaction.

**Duplica<sup>RealTime</sup> *Neisseria gonorrhoeae* 2<sup>nd</sup> Generation Detection Kit** is based on specific recognition and amplification of target sequences by PCR, and the simultaneous detection of the accumulation of PCR amplification products by fluorescent DNA probes. The probe designed to detect the target carries the fluorophore FAM (6-carboxy-fluorescein) at the 5' end, while the probe detecting the internal control (IC), is labelled with the fluorophore HEX (hexa-chloro-fluorescein). Both the probes have a non fluorescent quencher at the 3' end. If excited, the whole probe doesn't emit fluorescence, since the proximity of the quencher to the reporter prevents the emission of the fluorescence from the reporter (quenching effect).

#### REAGENTS PROVIDED

Each kit contains enough reagents to perform 32 tests when used in 4 analytical sessions with 6 **samples**, 1 **Positive Control of Amplification (Control 1, C1)**, and 1 **Reaction Blank (BM)** each.

#### Kit Components

Reagent	Color Code	Storage (range, °C)	Volume (µl)	Quantity (tubes)
<b>Oligo Mix (OM)*</b>	Green Cap	-22÷-18	400	1
<b>Amplification Mix (AM)</b>	Blue Cap	-22÷-18	400	1
<b>Control 1 (C1, Positive C.)</b>	Red Cap	-22÷-18	>50	1
<b>Control 2 (C2, Internal C.)</b>	Yellow Cap	-22÷-18	>50	1
<b>Reaction Blank (BM)</b>	White Cap	-22÷-18	>50	1

\*protect the tube from direct light

#### STORAGE AND HANDLING

All reagents must be stored at **-22÷-18°C** and can be used until the expiry date printed on the labels. Do not freeze and thaw the products more than six times.

## MATERIAL REQUIRED BUT NOT PROVIDED

- Extraction kit for DNA purification (refer to the specific handbook's section)
- Optical tubes or microplate for Real Time PCR
- Disposable powder-free gloves and laboratory coat
- Variable volumes pipettes (5-20 µl, 20-200 µl and 100-1000 µl)
- Disposable RNase/DNase-free tips with aerosol barriers
- Tube racks
- Desktop centrifuge
- PCR box
- Refrigerator
- Deep-freezer
- Thermalcycler for Real Time PCR

The kit has been developed to be used on SmartCycler® (Cepheid) and Rotor-Gene® Q (Qiagen) thermalcyclers. Other makes and models should be fully tested and evaluated for optimal performance by the user before reporting results. The equipments should be regularly maintained, in accordance with the manufacturer's instructions, and calibrated to ensure accurate PCR cycling and optimal performance.

## PRECAUTIONS AND WARNINGS

- In compliance with Good Laboratory Practice, define three separate laboratory's areas for: DNA extraction, PCR reaction mix preparation; manipulation of controls provided with the kit. Each area must have dedicated pipettes and laminar flow hood.
- If required, Euroclone Diagnostica offers the necessary technical support for the correct use of the kit.
- Carefully read this Instruction for Use before using the kit
- Do not use the reagents after the expiry date
- Thaw and carefully mix the reagents of the kit before use
- Do not mix the reagents from different lots of the product
- Use calibrated and regularly checked pipettes and instrumentation only
- Use dedicated laboratory equipments. Change gloves frequently
- Periodically wipe the working area with 0,5% hypochlorite
- Use powder-free gloves. Do not leave fingerprints on optical section of Smart Tubes.
- Materials containing or potentially-containing infectious agents must always be manipulated in a separated microbiological safety room under a Biohazard biological hood.
- In case of damaged package, contact the technical support before using the kit
- Do not use the product when stored at temperatures other than those indicated on the labels or described in this Instructions For Use.
- In case of spillage of the kit contents, please refer to the specific Material Safety Data Sheet (MSDS, available on request).
- The kit reagents, individual protective equipments, used materials, biological samples and test residuals must be disposed in accordance with local regulations.
- Patient Drug treatment may interfere with the final result of the molecular biology analysis.

## OPERATING PROCEDURE

### a) DNA purification

#### 1. DNA extraction for clinical specimens

- Collect the first morning urine into a clean, sealed container with no preservative.
- The specimens should be stored between 2÷8°C and ideally tested immediately or within 24 hours. Alternatively, the specimens should be frozen immediately upon receipt and stored up to ten days at -22÷-18°C or colder. A single freeze thaw cycle should not affect the results.
- EuroClone Diagnostica Srl recommend for Automatic extraction Duplica®Prep Body Fluid kit ref. EDI004200 and for Manual Extraction EDR004050 Bact Extra Pure Kit.

#### 2. DNA extraction for BD ProbeTec™ samples

- From previously lysed BD ProbeTec™ samples, please either use the DuplicaPrep Body Fluid kit according to the plasma protocol (ref. EDI004200) or Bact Extra Pure Kit (ref. EDR004050).

Other extraction reagents and methods should be fully tested and evaluated for optimal performance by the user before reporting results.

### b) Thermalcycler Setup

**Important points before starting:** Refer to the specific handbook of the equipment used to set the thermal profile indicated in the **Thermal Profile Table**. We recommend to switch on the instrument and to set the thermal profile before preparing the reaction mix.

*N.B.: before starting the run is recommended to save the file as "Neisseria gonorrhoeae Test"; in this way it is possible to save the Thermal Profile and settings and recall them in subsequent runs.*

### SmartCycler® platform

- Open the SmartCycler® software and click "Define Protocols".
- Click "New Protocol" (bottom left).
- Enter Name "**Neisseria gonorrhoeae Test**".
- When ready click "Save Protocol".

### Rotor-Gene® Q platform

- Start the software and on the box *New Run* select *Advanced*
- Select a *new template* in *Empty Run* or a pre-existing one
- Select the Rotor Type of your instrument and then *Next*
- Type 25 µl in the reaction volume and then *Next*
- Select *Edit Profile* and set up the correct Thermal Profile as indicated in the table below
- Select *Gain Optimisation* and then flag the option *Perform Optimisation before 1<sup>st</sup> acquisition*
- On *Channel Settings* select the green/yellow fluorophores and tube position "1" to perform the optimization. Then close the window and select *Next* and *Start Run*

### Thermal Profile Table (common for all the supported platforms)

TIME	TEMPERATURE	CYCLES
10 min	95°C	1
15 sec	95°C	45
60 sec	62°C	Fluorescence Acquisition

### c) PCR mix preparation

The total reaction volume is **25 µl**.

For each experiment prepare a PCR mix for the positive control (**C1**), the **Reaction Blank (BM)** and **n+1** samples. The internal control of amplification (**C2**) should be added directly to the PCR MIX, prior the aliquotation. The reagents of the PCR mix have to be mixed as indicated in the table below:

REAGENT	VOLUME (µl)
Amplification mix	10
Oligo mix	10
Internal Control (C2)	1
Extracted DNA	4

**The PCR mix has to be freshly prepared every time**

After its preparation, aliquot **21 µl of Master Mix** in the tubes or in the microplates well for PCR then add in each tube/well **4 µl** from the **extracted DNA or Control 1 DNA**, place in order the tubes/microplate in the instrument and start the program of amplification. At the end of the program remove the tubes/microplate from the thermalcycler.

### d) ANALYSIS and INTERPRETATION of RESULTS

**Important points before starting:** For a detailed description on how to analyze data, refer to *System User's manuals*. **Always visually inspect the amplification plot for each sample tested versus C<sub>T</sub> values obtained with the software.**

#### Results interpretation

Refer to the instrument-specific user guide to visualize the amplification plots for the entire plate/rotor. Detailed analysis of raw data depends on the real-time PCR instrument used. Baseline noise levels should either be set automatically or at predefined cycles.

The fluorescence in each channel indicates the hybridisation of the target specific probes:

- **Channel 1 for FAM/Green= Target probe**
- **Channel 2 for HEX/Cy3/Yellow= Internal Control probe.**
- If a sample shows a fluorescence in **FAM/Green (C<sub>T</sub>>0)**, the sample is surely positive and the signal detected by **HEX/Cy3/Yellow** fluorophore (C<sub>T</sub>≥0) is not relevant.
- When no signal at fluorophore **FAM/Green (C<sub>T</sub>=0)** is detected, to confirm the negative result, the completed amplification of internal control and therefore the appearance of a fluorescent signal at **HEX/Cy3/Yellow (C<sub>T</sub>>0)** fluorophore level, must be verified. Only in this case we can state that the sample is definitely negative.
- Condition in which no signal is detected indicates PCR inhibition. The sample must to be repeated and a dilution 1:10 of the target DNA is suggested.

Please be aware that:

- The Blank is positive in the **HEX/Cy3/Yellow** channel, but negative in the **FAM/Green** channel.
- The Positive Control is positive in both the **FAM/Green** and the **HEX/Cy3/Yellow** channels. The **HEX/Cy3/Yellow** signal, due to competition with **FAM/Green** probe, may arise later or be absent. In this case the **HEX/Cy3/Yellow** signal is not relevant and the run is valid.

**If all these conditions have been met, the run is valid and it's possible to analyse the data; otherwise the run is not valid. It's responsibility of the user to validate the run.**

### Results Interpretation Table

FAM/Green	HEX/ Cy3/Yellow	Results
$C_T > 0$	Not relevant ( $C_T \geq 0$ )	POSITIVE
$C_T = 0$	$C_T > 0$	NEGATIVE
$C_T = 0$	$C_T = 0$	INHIBITION

### TROUBLESHOOTING

#### **Problem 1: Weak or no signal of Positive Control, C1.**

1. The PCR conditions didn't comply with the instructions. All sample results are IVALID:
  - Check the amplification protocol and select the fluorescence channel reported in the manual.
2. Deterioration of dyes and/or primers. The reagents storage conditions didn't comply with the instructions:
  - Check storage conditions.

#### **Problem 2: Weak or no signal of the Internal Control, C2, in unknown samples AND Reaction Blank BM.**

1. The PCR was inhibited:
  - Make sure to use a recommended DNA purification method and carefully follow the manufacturer's instructions.
2. Pipetting error due to omitted reagents or samples:
  - Repeat the analysis starting from the PCR.
3. Deterioration of dyes and/or primers. The reagents storage conditions didn't comply with the instructions:
  - Check storage conditions.
4. Very low starting amount and/or low purity of genomic DNA. Improper DNA extraction:
  - Repeat the DNA purification.
5. Wrong channel/filter was chosen. The PCR conditions didn't comply with the instructions:
  - Check the PCR conditions and select the fluorescence channels reported in the protocol for the Unknown Sample detection.

#### **Problem 3: FAM signal in Reaction Blank BM.**

1. Contamination during PCR preparation procedure. All samples results are INVALID:
  - Decontaminate the working area and all instruments.
  - Pipette the positive control (C1) at last.
  - Repeat the PCR preparation using a new set of reagents.

#### **Problem 4: No FAM and HEX signals in unknown sample.**

1. The PCR was inhibited:
  - Make sure to use a recommended DNA purification method and carefully follow the manufacturer's instructions.

#### **Problem 5: Wide Fluctuations in fluorescence values.**

1. The PCR Master Mix is not well prepared:
  - Carefully repeat the PCR preparation procedure.
2. Air bubbles trapped in the PCR tubes:
  - Check the presence of air bubbles before starting a new run.

#### **Problem 6: Absence of any fluorescent signal.**

1. Verify the performance of the thermal cycler:
  - Calibrate the equipment.
2. Deterioration of dyes and/or primers. The storage conditions didn't comply with the instructions:
  - Check storage conditions.
  - Check the expiry date of the kit.

#### **Problem 7: The thermal cycler gives an error message.**

1. Refer to the real-time PCR instrument user manual or contact the local technical support of the real-time PCR instrument company.

#### **Problem 8: The kit reagents left out of the storage range temperature.**

1. These reagents must be stored as **indicated**, for a proper execution of the test. The performance of the product is not guaranteed if the reagents have not been properly stored.

**"THE PURCHASE OF THIS PRODUCT GRANTS THE PURCHASER RIGHTS UNDER CERTAIN ROCHE PATENTS TO USE IT SOLELY FOR PROVIDING HUMAN IN VITRO DIAGNOSTIC SERVICES. NO GENERAL PATENT OR OTHER LICENSE OF ANY KIND OTHER THAN THIS SPECIFIC RIGHT OF USE FROM PURCHASE IS GRANTED HEREBY"**

## Duplica<sup>RealTime</sup> *Neisseria gonorrhoeae* 2<sup>nd</sup> Generation Detection Kit

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