Duplica^{RealTime} FACTOR V H1299R Genotyping Kit

REV. EER010032_IFU_REV.00D_ENITA

REF: EER010032- 32 tests

Instructions For Use

INTENDED USE

Duplica^{RealTime} FACTOR V H1299R Genotyping Kit is an *in vitro* nucleic acid amplification test for the detection of the nucleotide substitution A4070G of the blood coagulation Factor V gene in human genomic DNA, extracted from peripheral whole blood samples collected in EDTA.

INTRODUCTION

Thrombophilia is a multigenic disorder caused by inherited and acquired defects. Individuals with multiple defects have a highly increased risk of development of Deep Venous Thrombosis (DVT). The A4070G (FV1299 His-Arg) polymorphism of the Factor V gene (FV; this allele is a part of R2 haplotype) in exon 13 was recently shown to influence circulating FV levels and to contribute to the Activated Protein C (APC) resistance phenotype. Heterozygous of both FV G1691A and FV A4070G substitutions is associated with a significant increasing risk of developing DVT (3-4 times if compared with the single FV R506Q substitution).

PRINCIPLE OF THE TEST

Duplica^{RealTime} FACTOR V H1299R Genotyping Kit is designed to identify the nucleotide substitution A4070G in the human Factor V gene. Two reaction mixes are provided for the amplification:

- **AMPLIFICATION MIX**, containing Hot Start Taq DNA polymerase, nucleotides, MgCl₂ and buffer.
- **OLIGO MIX**, containing primers and fluorogenic probes.

Duplica^{RealTime} FACTOR V H1299R Genotyping Kit is based on specific recognition and amplification of target sequences by PCR, and the simultaneous detection of the accumulation of PCR amplification products by fluorescent DNA probes. In particular, the probe designed to detect the Wild Type allele carries the fluorophore FAM (6-carboxy-fluorescein) at the 5' end, while the probe detecting the Mutated allele, is labelled with the fluorophore HEX (hexa-chloro-fluorescein). Both the probes have a non fluorescent black quencher at the 3' end. If excited, the whole probe doesn't emit fluorescence, since the proximity of the quencher to the reporter prevents the emission of the fluorescence from the reporter (quenching effect).

REAGENTS PROVIDED

Each kit contains enough reagents to perform 32 tests when used in 4 analytical sessions with 5 samples, 1 **Wild Type control (Control 1, C1)**, 1 **Mutated control (Control 2, C2)** and one **Reaction Blank (BM)** each

Kit Components

<i>Reagent</i>	<i>Color Code</i>	<i>Storage (range, °C)</i>	<i>Volume (µl)</i>	<i>Quantity (tubes)</i>
Oligo Mix (OM)*	Green Cap	-22÷-18	400	1
Amplification Mix (AM)	Blue Cap	-22÷-18	400	1
Control 1 (C1, Wild Type)	Red Cap	-22÷-18	>50	1
Control 2 (C2, Mutated)	Yellow Cap	-22÷-18	>50	1
Reaction Blank (BM)	White Cap	-22÷-18	>50	1

***protect the tube from direct light**

STORAGE AND HANDLING

All reagents must be stored at **-22÷-18°C** and can be used until the expiry date printed on the labels. Do not freeze and thaw the products more than six times.

MATERIAL REQUIRED BUT NOT PROVIDED

- Extraction kit for DNA purification (refer to the specific handbook's section)
- Optical tubes (Smart Tubes for SmartCycler® Real Time PCR platform)
- Disposable powder-free gloves and laboratory coat
- Variable volumes pipettes (5-20 µl, 20-200 µl and 100-1000 µl)
- Disposable RNase/DNase-free tips with aerosol barriers
- Tube racks
- Desktop centrifuge
- PCR box
- Refrigerator
- Deep-freezer
- Thermalcycler for Real Time PCR

The kit has been optimized to be used on SmartCycler® (Cepheid) and Rotor-Gene® Q (Qiagen) thermalcyclers. Other makes and models should be fully tested and evaluated for optimal performance by the user before reporting results.

The equipments should be regularly maintained, in accordance with the manufacturer's instructions, and calibrated to ensure an optimal performance.

PRECAUTIONS AND WARNINGS

- In compliance with Good Laboratory Practice, define three separate laboratory's areas for: DNA extraction, PCR reaction mix preparation; manipulation of controls provided with the kit. Each area must have dedicated pipettes and laminar flow hood
- If required, Euroclone Diagnostica offers the necessary technical support for the correct use of the kit.
- Carefully read this Instruction for Use before using the kit
- Do not use the reagents after the expiry date
- Thaw and carefully mix the reagents of the kit before use
- Do not mix the reagents from different lots of the product
- Use calibrated and regularly checked pipettes and instrumentation only
- Use dedicated laboratory equipments. Change gloves frequently

- Periodically wipe the working area with 0,5% hypochlorite
- Use powder-free gloves. Do not leave fingerprints on tubes and on optical part of Smart Tubes.
- Materials containing or potentially-containing infectious agents must always be manipulated in a separated microbiological safety room under a Biohazard biological hood
- In case of damaged package, contact the technical support before using the kit
- Do not use the product when stored at temperatures other than those indicated on the labels or described in this Instructions For Use
- In case of spillage of the kit contents, please refer to the specific Material Safety Data Sheet (MSDS, available on request)
- The kit reagents, individual protective equipments, used materials, biological samples and test residuals must be disposed in accordance with local regulations
- Patient Drug treatment may interfere with the final result of the molecular biology analysis

OPERATING PROCEDURE

a) DNA purification

For genomic DNA purification EuroClone Diagnostica recommends to use:

-Duplicα Blood DNA kit (ref. EDI002250) and Duplicα NA Body Fluid kit (ref. EDI004200) with Duplicα[®] PREP Automatic Extractor (ref. EDI001) for automated purification.

For manual extraction/purification:

-Fasst DNA Releaser (ref. EMR057050) for **fresh** peripheral blood (i.e. stored for up to 24 hours at 2÷8°C).

-Spin DNA Purification Kit (ref. EMR061050) for **frozen** blood or stored at 2÷8°C for more than 24 hours.

Other extraction reagents and methods should be fully tested and evaluated for optimal performance by the user before reporting results.

Attention! Use blood samples in EDTA anti-coagulant solution.

b) Thermalcycler Setup

We recommend to switch on the instrument and to set the Thermal Profile before preparing the reaction mix.

Important points before starting: Refer to the specific handbook of the equipment for detailed instructions on how to operate with the thermalcycler system and software version to set the Thermal Profile indicated in the **Thermal Profile Table**.

SmartCycler[®] platform

N.B.: before starting the run is recommended to save the file such as "FV H1299R"; this way it is possible to save the Thermal Profile and settings and recall them in subsequent runs.

- Open the SmartCycler[®] software and click "Define Protocols".
- Click "New Protocol" (bottom left).
- Enter Name, for example "FV H1299R".
- Enter the parameters as outlined in the Table below.

When ready click "Save Protocol".

Rotor-Gene[®] Q platform

- Start the software and on the box *New Run* select *Advanced*
 - Select a *new template* in *Empty Run* or a pre-existing one
 - Select the Rotor Type of your instrument and then *Next*
 - Type 25 µl in the reaction volume and then *Next*
 - Select *Edit Profile* and set up the correct Thermal Profile as indicated in the Table below
 - Select *Gain Optimisation* and then flag the option *Perform Optimisation before 1st acquisition*
- On *Channel Settings* select the green/yellow fluorophores and tube position "1" to perform the optimization. Then close the window and select *Next* and *Start Run*

Thermal Profile Table (common for both platforms)

Time	Temperature	Cycles
10 min	95°C	1
20 sec	95°C	50
60 sec	60°C	Fluorescence Acquisition

c) Preparation of PCR mix

The total reaction volume is **25 µl**.

For each experiment prepare a PCR mix for the **2 controls (C1 and C2)**, **1 Reaction Blank (BM)** and **n+1** samples. The reagents of the PCR mix have to be mixed as indicated in the table below:

REAGENT	VOLUME (µl)
Amplification mix	10
Oligo mix	10
Extracted DNA	5

The PCR mix has to be freshly prepared every time.

After its preparation, aliquot **20 µl of Master Mix** in the tubes or in the microplates well for PCR then add in each tube/well **5 µl** (100-250 ng/reaction) from the **extracted DNA** or **control DNA**, place in order the tubes/microplate in the instrument and start the program of amplification. At the end of the program remove the tubes/microplate from the thermalcycler.

ANALYSIS and INTERPRETATION of the RESULTS

Important points before starting: For a detailed description on how to analyze data, refer to the specific *System User's manual*. **Always visually inspect the amplification plot for each sample tested versus C_T values obtained with the software.**

Real Time PCR curves analysis

Refer to the instrument specific user guide to visualize the amplification plots for the entire plate/rotor. Detailed analysis of raw data depends on the Real Time PCR instrument used. Baseline noise levels should either be set automatically or at predefined cycles.

The fluorescence in each channel indicates the hybridisation of the allelic specific probes: **Channel 1 for FAM/green= Wild Type allele probe** and **Channel 2 for HEX/VIC/Yellow= Mutated allele probe**. If a sample shows a fluorescence in **FAM/Green**, the sample has the **Wild Type allele**. If a sample shows a fluorescence in **HEX/VIC/Yellow**, the sample has the **Mutated allele**.

Therefore, if only a **FAM/Green signal** is detected the sample is **Homozygous Wild Type**, whereas if only a **HEX/VIC/Yellow signal** is detected the sample is **Homozygous Mutated**. Finally, if both **FAM/Green** and **HEX/VIC/Yellow** are detected the sample is **Heterozygous**.

Condition in which no signal is detected indicates PCR inhibition. In this case, refer to troubleshooting.

Control 1 (Wild Type), Control 2 (Mutant) and reaction blank (BM) are provided in order to properly set the threshold line before samples analysis. After the run, the threshold line has to be set so that: **Control 1 is detected in FAM/Green and not detected in HEX/VIC/Yellow, whereas Control 2 is detected in HEX/VIC/Yellow and not detected in FAM/Green. The Reaction Blank must not be detected in any channel.**

If these three conditions have been met, the run is valid and it's possible to analyse the data; otherwise the run is not valid. It's responsibility of the user to validate the run.

Results Interpretation Table

FAM/Green	HEX/VIC/Yellow	Results
Detected	Not detected	Wild Type
Not detected	Detected	Mutated
Detected	Detected	Heterozygous
Not detected	Not detected	Inhibition

TROUBLESHOOTING

Problem 1: Weak or no signal of unknown samples.

1. The PCR was inhibited:
 - Make sure to use a recommended DNA purification method and carefully follow the manufacturer's instructions.
2. Pipetting error due to omitted reagents or samples:
 - Repeat the analysis starting from the PCR.
3. Deterioration of dyes and/or primers. The reagents storage conditions didn't comply with the instructions:
 - Check storage conditions.
4. Very low starting amount and/or low purity of genomic DNA. Improper DNA extraction:
 - Repeat the DNA purification.
5. Wrong channel/filter was chosen. The PCR conditions didn't comply with the instructions:
 - Check the PCR conditions and select the fluorescence channels reported in the protocol for the Unknown Sample detection.

Problem 2: Weak or no signal of the Wild Type Control C1 and/or Mutated Control C2.

1. The PCR conditions didn't comply with the instructions:
 - Check the amplification protocol and select the fluorescence channel reported in the manual.
2. Deterioration of dyes and/or primers. The reagents storage conditions didn't comply with the instructions:
 - Check storage conditions.

Problem 3: Any signal with Reaction Blank BM.

1. Contamination during PCR preparation procedure. All samples results are INVALID:
 - Decontaminate the working area and all instruments.
 - Pipette the controls C1 and C2 at last.
 - Repeat the PCR preparation using a new set of reagents.

Problem 4: HEX signal in Wild Type Control C1 and/or FAM signal in Mutated Control C2.

1. Contamination during PCR preparation procedure. All samples results are INVALID:
 - Decontaminate the working area and all instruments.
 - Use tips with aerosol barriers only. Change tips between tubes/wells.
 - Repeat the PCR preparation using a new set of reagents.

Problem 5: Fluorescence intensity varies.

1. The PCR Master Mix is not well prepared:
 - Carefully repeat the PCR preparation procedure.
2. Air bubbles trapped in the PCR tubes:
 - Check the presence of air bubbles before starting a new run.

Problem 6: Absence of any fluorescent signal.

1. Verify the performance of the thermalcycler:
 - Calibrate the equipment.
2. Deterioration of dyes and/or primers. The storage conditions didn't comply with the instructions:
 - Check storage conditions.
 - Check the expiry date of the kit.

Problem 7: The thermalcycler gives an error message.

1. Refer to the Real Time PCR instrument user manual or contact the local technical support of the Real Time PCR instrument company.

Problem 8: The kit reagents left out of the storage range temperature.

1. These reagents must be stored as indicated for a proper execution of the test. The performance of the product is not guaranteed if the reagents have not been properly stored.

"THE PURCHASE OF THIS PRODUCT GRANTS THE PURCHASER RIGHTS UNDER CERTAIN ROCHE PATENTS TO USE IT SOLELY FOR PROVIDING HUMAN IN VITRO DIAGNOSTIC SERVICES. NO GENERAL PATENT OR OTHER LICENSE OF ANY KIND OTHER THAN THIS SPECIFIC RIGHT OF USE FROM PURCHASE IS GRANTED HEREBY"

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EuroClone Diagnostica Srl