

RAPID DIAGNOSTICS

# osom<sup>®</sup>

## Mono Test

TO AID IN THE DETECTION OF MONONUCLEOSIS



[www.osomtests.com](http://www.osomtests.com)



### FEATURES:

- ✓ **ACCURATE;** > 99% sensitivity;  
96% specificity
- ✓ **FAST;** Results in 5 minutes
- ✓ **SAMPLE;** Whole blood, serum or plasma
- ✓ **NO AGE LIMITATIONS**
- ✓ **QC INSIDE;** Includes 2 additional tests for external QC
- ✓ **TRAINING;** Online training and certification; [www.osomtraining.com](http://www.osomtraining.com)
- ✓ **CUSTOM CARE;** Technical support staffed with certified medical technologists



# OSOM<sup>®</sup>

## Mono Test

**Sekisui Diagnostics, LLC (AMERICAS)**

4 Hartwell Place Lexington, MA 02421

Phone: 800 332 1042

Fax: 800 762 6311

Email: [osomrapidtests@sekisui-dx.com](mailto:osomrapidtests@sekisui-dx.com)

ISO 13485 Certified

**Sekisui Diagnostics (UK) Limited (INTERNATIONAL)**

Liphook Way, Allington, Maidstone, Kent ME16 0LQ, UK

Phone: +44 1622 607800 Fax: +44 1622 607801

Email: [info@sekisui-dx.com](mailto:info@sekisui-dx.com)

ISO 9001 Certified, ISO 13485 Certified

TEST PROCEDURE / RESULTS



For serum, plasma or whole blood from collection tubes, add 1 drop to test tube from pipette.

For fingertip blood, fill capillary tube and dispense blood into test tube.




Add 1 drop of diluent to test tube. Mix.

Place the test stick into the treated sample.



Read results at **5 MINUTES.**

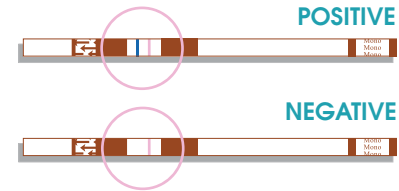

DESCRIPTION

OSOM<sup>®</sup> MONO TEST

CONFIGURATION

25 TESTS + 2 TESTS FOR QC

CATALOG #

145 (CE MARKED)

STORAGE

ROOM TEMP

CPT CODE

86308QW (WHOLE BLOOD)  
86308 (SERUM, PLASMA)



MADE IN THE USA