Duplic^{RealTime} MDR A.baumannii OXA Genotyping Kit

REV. EBR040032_IFU_REV.00D_ENITA

REF: EBR040032- 32 tests

Instructions For Use

INTENDED USE

Duplic^{RealTime} MDR A.baumannii OXA Genotyping Kit is an in vitro qualitative assay for the detection and identification, starting from culture specimens, of the most common OXA genes conferring antibiotic resistance to *Acinetobacter baumannii*: bla_{OXA-58}, bla_{OXA-23}, bla_{OXA-24}, bla_{OXA-143} genes and the ISAb_{a1}-bla_{OXA-51}-like junction.

INTRODUCTION

Multidrug-resistant *Acinetobacter baumannii* is recognized to be one of the most hardly-treatable and controllable antimicrobial-resistant Gram-negative bacilli. Increasing antimicrobial resistance among *Acinetobacter* isolates has been documented, although several definitions of multidrug resistance are reported in the literature. *A.baumannii* survives for prolonged periods under a wide range of environmental conditions. The organism causes outbreaks of infection and health care-associated infections, including bacteremia, pneumonia, meningitis, urinary tract infection and wound infection.

The management of this pathogen has become a significant challenge due to the increased emergence of carbapenem-resistant strains. It has been shown that most of the carbapenem resistance in *A.baumannii* is due to the production of carbapenemases, especially those belonging to Carbapenem-Hydrolyzing class D beta-Lactamases (CHDLs), which are encoded by the bla_{OXA-51}-like, bla_{OXA-23}-like, bla_{OXA-24}-like, and bla_{OXA-58}-like genes and the emerging bla_{OXA-143}-like gene. Among these CHDL genes, the bla_{OXA-51}-like gene is intrinsic to and is located on the chromosome of *A.baumannii*, while the other bla_{OXA} genes are normally located on extrachromosomal plasmids and can migrate from one strain to another, conferring resistance to the host strains.

The CHDLs have a weak carbapenem-hydrolyzing activity. However, the overexpression of these CHDL genes, mostly driven by promoters provided by their upstream Insertion Sequences (ISs), is one of the means by which *A.baumannii* acquires a high level of carbapenem resistance. For example ISAb_{a1}, upstream of the bla_{OXA-51}-like gene, is associated with the overexpression of the bla_{OXA-51}-like gene and the carbapenem resistance in *A.baumannii*, while the other bla_{OXA} genes are normally overexpressed since they are located on plasmids that are usually present in multiple copies.

Since the therapeutic options for the infection of *Acinetobacter* are limited, the development or discovery of new therapies, as well as methods that allow the analysis and identification of this bacterium in dedicated care facilities, are crucial to limit and prevent its spread. This kit offer the opportunity to identify these major OXA bla-genes as well as ISAb_{a1}-bla_{OXA51} junction.

PRINCIPLE OF THE TEST

Duplic^{RealTime} MDR A.baumannii OXA Genotyping Kit has been designed to identify the most common OXA genes conferring antibiotic resistance to *Acinetobacter baumannii* using two distinct tests for each sample. The reagents for the amplification are ready to use and provided with 3 reactions mixes:

- **AMPLIFICATION MIX** containing Hot Start Taq DNA polymerase (modified Antibody technology), nucleotides, MgCl₂ and buffer.
- **OLIGO MIX A** containing primers and fluorogenic probes to detect bla_{OXA-51}, bla_{OXA-24} genes, the ISAb_{a1}-bla_{OXA-51}-like junction and the Internal Control DNA.
- **OLIGO MIX B** containing primers and fluorogenic probes to detect bla_{OXA-58}, bla_{OXA-23}, bla_{OXA-143} gene

and the Internal Control DNA.

Duplic^{RealTime} MDR A.baumannii OXA Genotyping Kit is based on specific recognition and amplification of target sequences by PCR, and the simultaneous detection of the accumulation of PCR amplification products by fluorescent DNA probes. A control DNA molecule, the Internal Control (CI), is added to the clinical specimen prior to DNA purification to monitor that DNA extraction and PCR amplification were successful. Seven Taqman probes, labeled with 4 different dyes are used to detect the various OXA-genes and the control DNA.

The Duplic^{RealTime} MDR A.baumannii OXA Genotyping Kit discriminates between bla_{OXA-58}, bla_{OXA-23}, bla_{OXA-24}, bla_{OXA-143} genes and the ISAb₁-bla_{OXA-51}-like junction. Moreover the kit allows to identify the A.baumannii strain respect the Acinetobacter strains due to the recognition of bla_{OXA-51} gene.

REAGENTS PROVIDED

The kit is provided in two different boxes (as reported in the following table). It has been designed to carry out 32 reactions. This format allows to perform for example 4 analytical sessions, each of which includes: 6 samples, 2 Controls (Positive Control, C+ and Negative Control, C-). The Internal Control provided must be added before the extraction to each sample (see paragraph OPERATING PROCEDURE) to identify any inhibition in the amplification reaction.

Kit Components

Reagent	Color Code	Box	Storage (Range, °C)	Volume (µl)	Quantity (tubes)
Oligo Mix A (OM A)*	Green Cap	1	-22 ÷ -18	400	1
Oligo Mix B (OM B)*	Pink Cap	1	-22 ÷ -18	400	1
Amplification Mix (AM)	Blue Cap	1	-22 ÷ -18	800	1
Positive Control (C+)	Red Cap	2	-22 ÷ -18	>50	1
Negative Control (C-)	White Cap	2	-22 ÷ -18	>50	1
Internal Control (CI)	Yellow Cap	2	-22 ÷ -18	>50	1

* protect the tubes from direct light

STORAGE AND HANDLING

All reagents have to be stored at -22 ÷ -18 °C and can be used until the expiry date printed on the labels. Do not freeze and thaw the products more than six times.

MATERIAL REQUIRED BUT NOT PROVIDED

- Optical tubes or microplate for Real Time PCR platform
- Disposable powder-free gloves and laboratory coat
- Variable volumes pipettes (5-20 µl, 20-200 µl, 100-1000 µl)
- Disposable tips with DNase/RNase-free aerosol barriers
- Tube racks
- Desktop centrifuge
- Vortex
- PCR box
- Refrigerator
- Deep-freezer
- Heating block
- Thermocycler for Real Time PCR

The kit has been developed to be used on Rotor-Gene[®] Q (Qiagen), DX[®]/CFX (Bio-Rad) and Applied Biosystems[®] 7500 (Life Technologies[™]) thermalcyclers. Other makes and models should be fully tested and evaluated for optimal performance by the user before reporting results.

All the equipments should be regularly maintained in accordance with manufacturer's instructions and calibrated to ensure an optimal performance.

PRECAUTIONS AND WARNINGS

- In compliance with Good Laboratory Practice, define three separate laboratory's areas for: DNA extraction, PCR reaction mix preparation; manipulation of controls provided with the kit. Each area must have dedicated pipettes and laminar flow hood.
- If required, Euroclone Diagnostica offers the necessary technical support for the correct use of the kit.
- Carefully read this Instruction for Use before using the kit
- Do not use the reagents after the expiry date
- Thaw and carefully mix the reagents of the kit before use
- Do not mix the reagents from different lots of the product
- Use calibrated and regularly checked pipettes and instrumentation only
- Use dedicated laboratory equipments. Change gloves frequently
- Periodically wipe the working area with 0,5% hypochlorite
- Use powder-free gloves. Do not leave fingerprints on optical section of Tubes.
- Materials containing or potentially-containing infectious agents must always be manipulated in a separated microbiological safety room under a Biohazard biological hood
- In case of damaged package, contact the technical support before using the kit
- Do not use the product when stored at temperatures other than those indicated on the labels or described in this Instructions For Use
- In case of spillage of the kit contents, please refer to the specific Material Safety Data Sheet (MSDS, available on request)
- The kit reagents, individual protective equipments, used materials, biological samples and test residuals must be disposed in accordance with local regulations
- Patient Drug treatment may interfere with the final result of the molecular biology analysis

OPERATING PROCEDURE

a) Crude DNA purification from bacterial cells

Procedure:

- Inoculate nutrient agar plates with the clinical samples or the bacterial strains to be tested and incubate overnight at 37 °C.
- Prepare a bacterial cell suspension of McFarland 0,5-1,0 using PCR-grade water.
- For each cell suspension, transfer 200 µl to a 1.5 mL Eppendorf tube and add 10 µl of internal control solution (CI solution, Yellow Cap). Mix briefly.
- Heat the tubes at 98°C for 10 minutes. After incubation vortex the tubes vigorously for 30 seconds.
- Use the suspension immediately or store at +4°C and use within 24 hours.

b) Thermalcycler Setup

Important points before starting: Refer to the specific handbook of the equipment used to set the thermal profile indicated in the Thermal Profile Table. We recommend to switch on the instrument and to set the thermal profile before preparing the reaction mix.

N.B.: before starting the run is recommended to save the file as "A. baumannii Test"; this way it is possible to save the Thermal Profile and settings and recall them in subsequent runs.

Rotor-Gene® Q platform

- Start the software and on the box New Run select Advanced
- Select a new template in Empty Run or a pre-existing one
- Select the Rotor Type of your instrument and then Next
- Type 25 µl in the reaction volume and then Next
- Select Edit Profile and set up the correct Thermal Profile as indicated in the table below
- Select Gain Optimisation and then flag the option Perform Optimisation before 1st acquisition
- On Channel Settings select the green/yellow/orange and red fluorophores and tube position "1" to perform the optimization. Then close the window and select Next and Start Run

DX®/CFX platform

- Select Create a new Experiment
- In the Protocol section select Create New to set the thermal profile
- In Plate Editor: set FAM, HEX, TEXAS RED and Cy5 as Fluorophores and define the sample name.

Applied Biosystems® 7500 platform

- Select Create a new Experiment

- In the Experiment Properties select Quantitation-Standard curve as experiment type and Taqman[®] Reagents as reagent type
- Select the Run Mode (7500 or 7500 Fast) and the Ramp Speed (Standard)
- In the Setup menu select Run Method to set the thermal profile
- In the Setup menu select Plate Setup-Define Targets and Samples to select Samples and assign FAM, VIC, TEXAS RED and Cy5 to targets and positive control (C+). Assign Cy5 fluorophore to internal control (CI) and negative control (C-). Leave the default value (NFQ-MGB) as quencher type
- In Define Samples insert the sample name
- Select the used wells in Plate Setup-Assign Targets and Samples
- Select None as passive reference

Thermal Profile Table (common for all the supported platforms)

TIME	TEMPERATURE	CYCLES
5 min	95°C	1
15 sec	95°C	35
60 sec	60°C	Fluorescence Acquisition

c) PCR mix preparation

Each sample has to be tested with two different mixes (A and B). For each PCR run prepare 2 PCR mixes one containing oligonucleotides and probes belonging to Mix A (OM A) and one containing oligonucleotides and probes belonging to Mix B (OM B). Every PCR has to contain the Positive Control (C+) and the Negative Control (C-). Prepare the Real-Time PCR mixes as described below: multiply the Amplification Mix and the Oligo Mix volumes per test by the total number of samples (including controls) and add a 10% surplus to ensure that you have enough reaction mix for all the calculated reactions.

How to prepare MIX A	
REAGENT	VOLUME for one sample (µl)
Amplification Mix (AM)	10
Oligo Mix A (OM A)	10
Purified DNA	5
Total Volume	25
How to prepare MIX B	
REAGENT	VOLUME for one sample (µl)
Amplification Mix (AM)	10
Oligo Mix B (OM B)	10
Purified DNA	5
Total Volume	25

The PCR mix has to be freshly prepared every time.

After its preparation, aliquot 20 µl of Master Mix in the PCR tubes, then add in each tube 5 µl of sample or control DNA.

d) ANALYSIS and INTERPRETATION of RESULTS

Important points before starting: For a detailed description on how to analyze data, refer to System User's manuals. Always visually inspect the amplification plot for each sample tested versus C_T values obtained with the software.

PRE-ANALYSIS SETUP

Rotor-Gene[®] Q platform

1. Carry out the analysis of the data with "Quantitation Analysis": this analysis allows to observe the individual curves of the different fluorophores.
2. Select "Slope Correct", follow the curves in linear scale and set "Outlier Removal" at 15%.
3. For each fluorophore, set the threshold value equal to 0.1.

DX[®] / CFX platform

1. In the Settings menu set the curve analysis in "Regression" mode.

Applied Biosystems[®] 7500 platform

- In the Analysis menu set the following threshold values:
 - FAM, VIC and Texas Red at 100,000.
 - Cy5 at 50,000.
- For each fluorophore, set the baseline value between 3 and 15

Results interpretation

Refer to the instrument-specific user guide to visualize the amplification plots for the entire plate/rotor. Detailed analysis of raw data depends on the real-time PCR instrument used. Baseline noise levels should either be set automatically or at predefined cycles.

The fluorescence in each channel indicates the hybridisation of the specific probes:

- Channel 1 for FAM/Green= Target probe
- Channel 2 for HEX/VIC/Yellow= Target probe
- Channel 3 for TEXAS RED/Orange= Target probe
- Channel 4 for Cy5/Red= Internal Control probe
- If a sample shows a fluorescence in FAM/Green, HEX/VIC/Yellow, TEXAS RED/Orange ($C_T > 0$), the sample is surely positive and the signal detected by fluorophore Cy5/Red ($C_T = 0$) is not relevant.
- When no signal at fluorophore FAM/Green, HEX/VIC/Yellow, TEXAS RED/Orange ($C_T = 0$) is detected, to confirm the negative result, the completed amplification of internal control and therefore the appearance of a fluorescent signal at Cy5/Red ($C_T > 0$) fluorophore level, must be verified. Only in this case we can state that the sample is definitely negative.
- Condition in which no signal is detected indicates PCR inhibition. The sample must to be repeated and a dilution 1:10 of the target DNA is suggested.

If all these conditions have been met, the run is valid and it's possible to analyse the data; otherwise the run is not valid. It's responsibility of the user to validate the run.


Run validation

Check the positive and negative control amplification curves. Valid runs (see the following pictures, belonging to each validated platform, as examples of positive results) show:


- No instruments system failures during the run.
- Positive and Negative Controls are within the C_T values specified in the following tables.

MIX A				
Controls /Expected C_T values	bla _{Oxa51} FAM/Green	IsAba1-OXA-51 Junction HEX/VIC/Yellow	bla _{Oxa24} TEXAS RED/Orange	InternalControl (CI) Cy5/Red
Positive Control (C+)	22 ±3	22 ±3	22 ±3	N/A
Negative Control (C-)	N/A	N/A	N/A	22±3
MIX B				
Controls /Expected C_T values	bla _{Oxa23} FAM/Green	bla _{Oxa58} HEX/VIC/Yellow	bla _{Oxa143} TEXAS RED/Orange	Internal Control (CI) Cy5/Red
Positive Control (C+)	22 ±3	22±3	22 ±3	N/A
Negative Control (C-)	N/A	N/A	N/A	22±3

Rotor-Gene® Q platform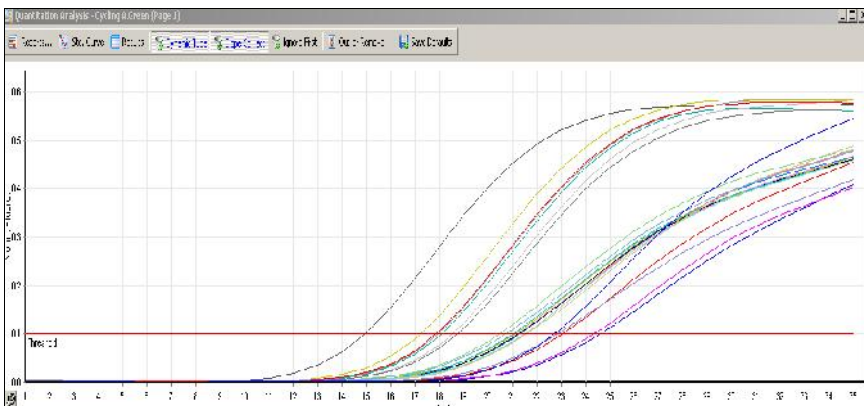

DX[®] / CFX platform

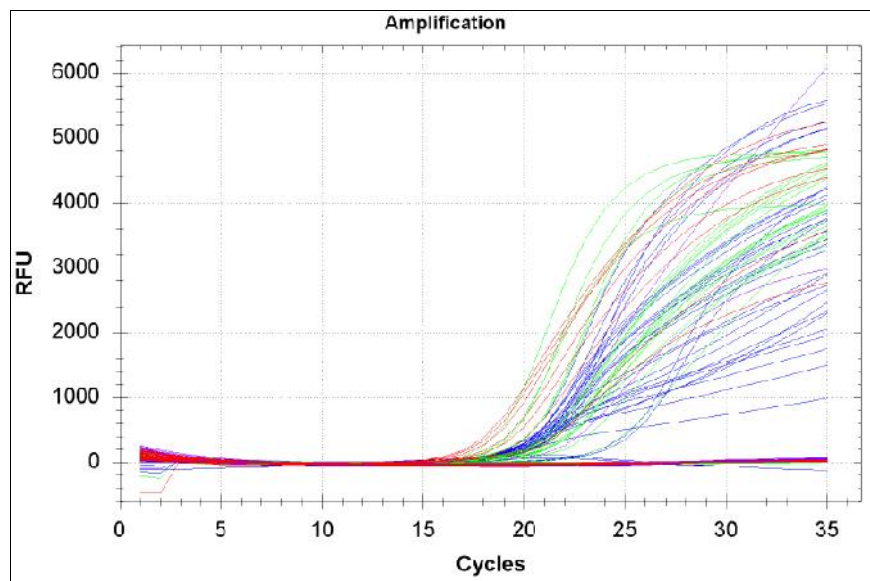Applied Biosystems[®] 7500 platform

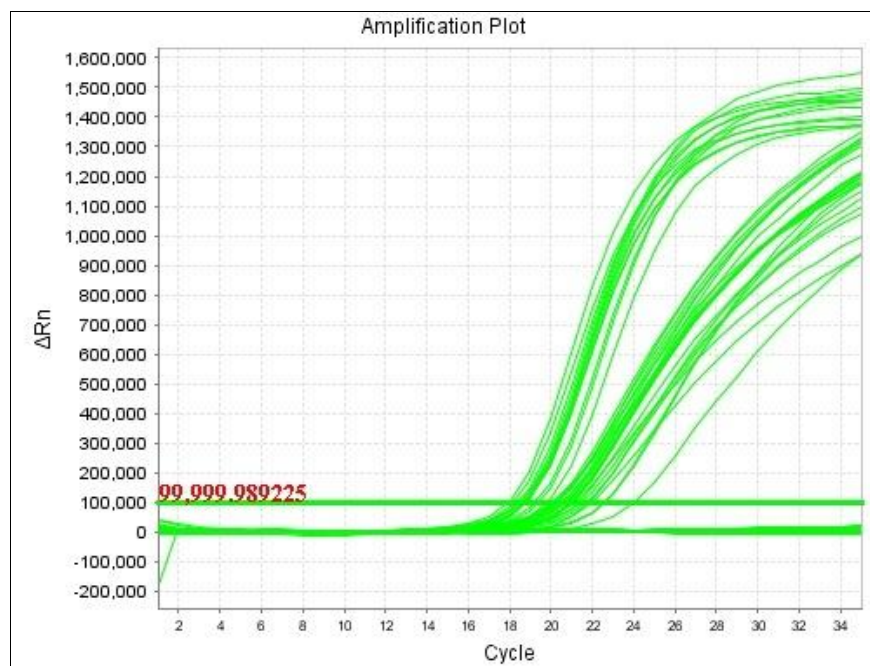
Fig: Examples of positive results obtained on different platforms. In the Applied Biosystem platform, the examples belong to bla_{OXA-51} and bla_{OXA-23} species.

Results interpretation

- OXA-negative *A.baumannii* samples will show a C_T value in the FAM/Green (bla_{OXA51} detected) channel of mix A and a C_T ≤25 in the Cy5/Red channel (Internal Control detected) of mix B. Detection of the Internal Control (Cy5/Red channel) in mix A is not relevant.
- OXA-positive *A.baumannii* samples will show a C_T value in the FAM/Green (bla_{OXA51} detected) channel of mix A and at least one among the HEX/VIC/Yellow (ISAb1-bla_{OXA51}junction detected), TxR/Orange (bla_{OXA24} detected) channels of mix A and/or FAM/Green (bla_{OXA23} detected), HEX/VIC/Yellow (bla_{OXA58} detected), TxR/Orange (bla_{OXA143} detected) channels of mix B. Detection of the Internal Control (a C_T ≤25 in the Cy5/Red channel) is mandatory when no signal is detected for all of the other three channels (FAM/Green, HEX/VIC/Yellow and TxR/Orange) of mix B, while it is not relevant in mix A or if one of the

other channels is detected in mix B.

- OXA-negative strains not belonging to the *A.baumannii* species (not containing the *bla*_{OXA51} gene) will only show a $C_T \leq 25$ in the Cy5/Red (Internal Control detected) channel of both mixes.
- OXA-positive strains not belonging to the *A.baumannii* species (not containing the *bla*_{OXA51} gene) will show a C_T value in at least one among the TxR/Orange (*bla*_{OXA24} detected) channel of mix A and/or FAM/Green (*bla*_{OXA23} detected), HEX/VIC/Yellow (*bla*_{OXA58} detected), TxR/Orange (*bla*_{OXA143} detected) channels of mix B. Detection of the Internal Control (a $C_T \leq 25$ in the Cy5/Red channel) is mandatory when no signal is detected for all of the other three channels (FAM/Green, HEX/VIC/Yellow and TxR/Orange) of both mixes.

RESULTS TABLE *					
<i>bla</i> _{OXA51} detected	ISAbal- <i>bla</i> _{OXA51} junction detected	<i>bla</i> _{OXA23} , <i>bla</i> _{OXA24} , <i>bla</i> _{OXA58} , <i>bla</i> _{OXA143}	Internal Control detected		Interpretation
			Mix A	Mix B	
Yes	No	No	Yes or No	$C_T \leq 25$	OXA-Negative <i>A. baumannii</i>
Yes	Yes (at least one)		Yes or No	$C_T \leq 25$ only if negative for <i>bla</i> _{OXA23} , <i>bla</i> _{OXA58} and <i>bla</i> _{OXA143}	OXA-Positive <i>A. baumannii</i>
No	No		$C_T \leq 25$	$C_T \leq 25$	OXA-Negative not <i>A. baumannii</i>
No	No	Yes (at least one)	$C_T \leq 25$ only if negative for <i>bla</i> _{OXA24}	$C_T \leq 25$ only if negative for <i>bla</i> _{OXA23} , <i>bla</i> _{OXA58} and <i>bla</i> _{OXA143}	OXA-Positive not <i>A. baumannii</i>
No	No	No	$C_T \leq 25$ in Mix A and/or Mix B		INVALID

* If observed C_T values vary significantly from expected C_T values, see Troubleshooting section

TROUBLESHOOTING

Problem 1: Weak or no signal of the Positive Control (C+).

1. The PCR conditions didn't comply with the instructions. All sample results are INVALID:
 - The positive controls solution was not added. Repeat the test
 - Amplification mix or oligo mixes were not added to the assay Please repeat the test.
 - Check the amplification protocol and select the fluorescence channels reported in the manual.
2. Deterioration of dyes and/or primers. The reagents storage conditions didn't comply with the instructions.
 - Check storage conditions.
 - Check the expiration date

Problem 2: Weak or no Cy5 signal (CI) in samples showing no amplification in the other 3 channels of one of the two test tubes

1. The PCR was inhibited. All sample results are INVALID:
 - Make sure that you use the recommended DNA purification method and carefully follow the instructions reported in this manual.
 - Repeat the test using the same purified DNA. If the invalid results persist, repeat the test by preparing a new DNA purified from original specimen. Alternatively, repeat the test with a new DNA extraction from a newly collected specimen.
2. Pipetting error due to omitted reagent:
 - Repeat the analysis starting from the PCR.
3. Deterioration of dyes and/or primers. The reagents storage conditions didn't comply with the instructions.
 - Check storage conditions:
 - Check the expiration date
4. Wrong channel/filter was chosen. The PCR conditions didn't comply with the instructions:
 - Check the PCR conditions and select the fluorescence channels reported in the protocol for the Unknown Sample detection.

Problem 3: Weak or no Cy5 signal (CI) in the Negative Control (C-)

1. The PCR conditions didn't comply with the instructions. All sample results are INVALID:
 - The positive controls solution was not added. Repeat the test

- Amplification mix or oligo mixes were not added to the assay Please repeat the test.
 - Check the amplification protocol and select the fluorescence channels reported in the manual.
2. Deterioration of dyes and/or primers. The reagents storage conditions didn't comply with the instructions:
- Check storage conditions.
 - Check the expiration date

Problem 4: FAM, Cy3 and/or TXR signals in the Negative Control (C-).

1. Contamination during PCR preparation procedure. All samples results are INVALID:
- Decontaminate the working area and all instruments.
 - Pipette the Positive control at last.
 - Repeat the PCR preparation using a new set of reagents.

Problem 5: Wide Fluctuations in fluorescence values.

1. The PCR Master Mix is not well prepared:
- Carefully repeat the PCR preparation procedure.
2. Air bubbles trapped in the PCR tubes:
- Check the presence of air bubbles before starting a new run.

Problem 6: Absence of any fluorescent signal.

1. Verify the performance of the thermalcycler:
- Calibrate the equipment.
2. Deterioration of dyes and/or primers. The reagents storage conditions didn't comply with the instructions:
- Check storage conditions

Problem 7: The Real Time PCR instrument gives an error message.

1. Refer to the real-time PCR instrument user manual or contact the local technical support of the real-time PCR instrument company.

Problem 8: Duplicate samples with Duplicon^{RealTime} MDR A. baumannii OXA Genotyping test did not yield identical results.

1. C_T values of identical samples may vary between individual reactions. Large variations, >2 C_T values, suggest pipetting errors or other differences between the duplicate samples.

Problem 9: Solutions (amplification mix, oligo mixes, Internal control, negative, or positive control) left out of the -22 ÷ -18 °C storage range.

1. These reagents must be stored at -22 ÷ -18 °C for proper performance of the test. The performance of the product cannot be guaranteed if these solutions were left out of -22 ÷ -18 °C.

"THE PURCHASE OF THIS PRODUCT GRANTS THE PURCHASER RIGHTS UNDER CERTAIN ROCHE PATENTS TO USE IT SOLELY FOR PROVIDING HUMAN IN VITRO DIAGNOSTIC SERVICES. NO GENERAL PATENT OR OTHER LICENSE OF ANY KIND OTHER THAN THIS SPECIFIC RIGHT OF USE FROM PURCHASE IS GRANTED HEREBY"

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EuroClone Diagnostica Srl